


# It's Neo-Hydro /



Neo-hydro Multi-purpose Drainage Catheter Removal\*


## Step 1

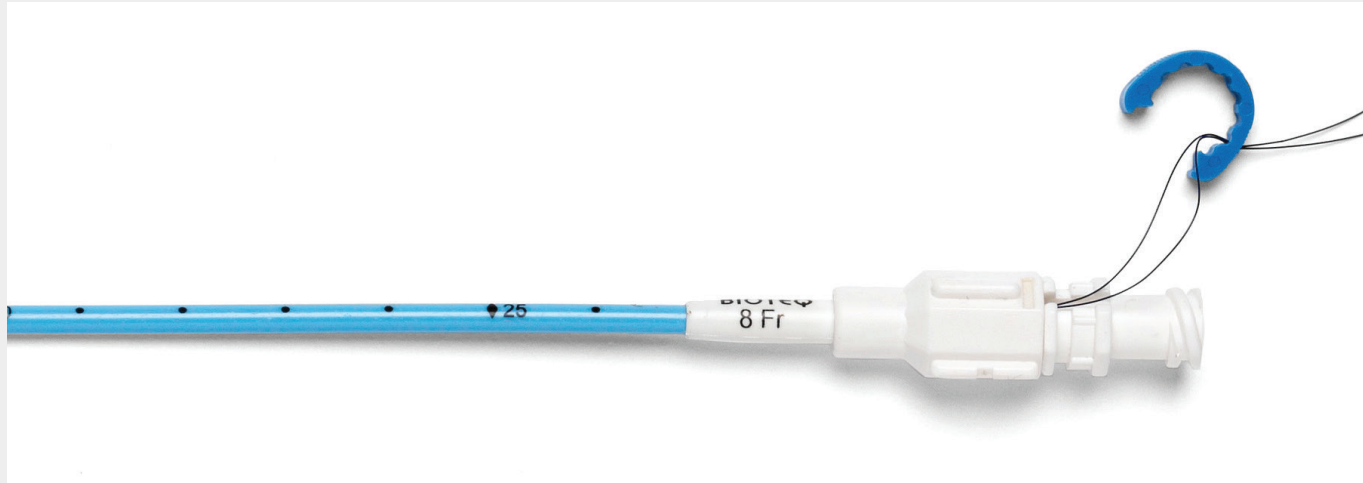

Remove the blue clip and unwind the draw string.


Please Note:  
This document is intended as a visual reference guide only. Prior to using this product, please read the full manufacturer's instructions for use included in each box of Neo-hydro multi-purpose drainage catheters.

## Step 2

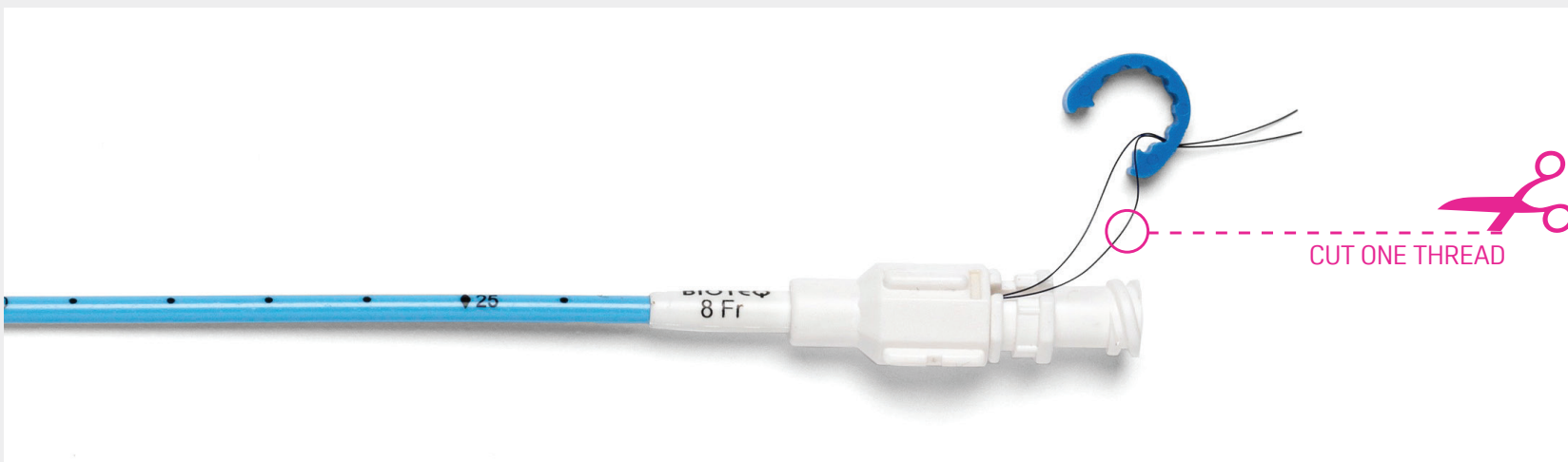

Check that both threads are loose and then cut one thread in order to loosen the pigtail.

## Step 3

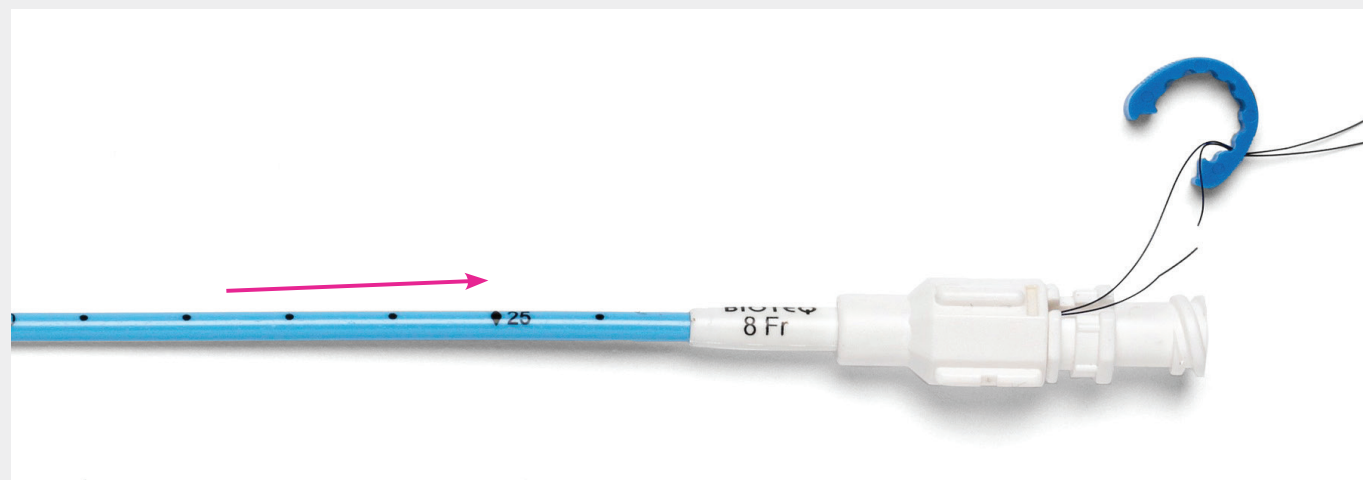

Holding the blue collar and catheter hub slowly withdraw the catheter. The loose end of the thread should retract into the hub as the pigtail straightens. If there is any bleeding post removal it may be appropriate to apply firm pressure and place a dressing over the puncture site.

\*Please Note: A non-locking Neo-hydro will not have a clip or draw string and can be removed by gently pulling the catheter from the patient. If access is to be maintained, pass a straight floppy-tip guidewire through the catheter. This may also facilitate removal.