


# It's Neo-Hydro (Non-Locking) /

## Steps for use

These instructions are to be used as a reference guide only. Please refer to the full instructions for use prior to using this product.

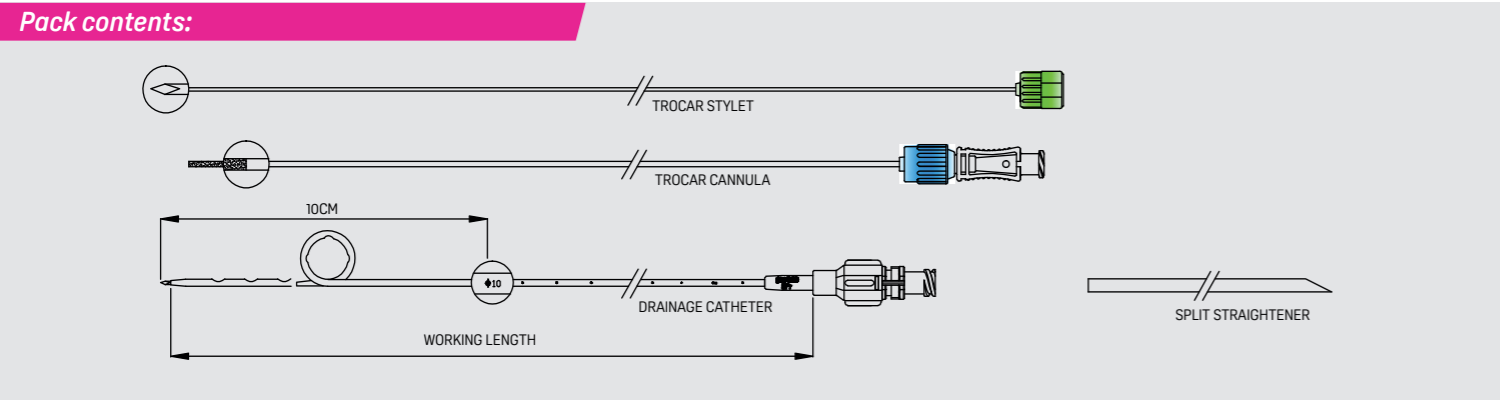

**Step 1.**

Place the pigtail straightening device over the pigtail.

**Step 2.**

Feed the cannula (blue hub) through catheter and ensure the luer lock is secured.

**Step 3.**

Remove the pigtail straightening device.\*

**Step 4.**

Remove the paper holder, advance the stylet and screw the green hub to the cannula.

**Step 5.**

Ensure the stylet tip is exposed beyond the catheter. The catheter is now ready to be inserted.

**Step 6.**

Unscrew the green luer lock hub and withdraw the stylet.

**Step 7.**

Attach a luer lock syringe to the clear plastic hub and aspirate fluid.\*\*

**Step 8.**

Hold the catheter in place & unscrew the blue luer lock cannula hub. Keep the metal cannula still & slowly advance the catheter until you are satisfied with its position (arrow 1). Now withdraw the metal cannula (arrow 2).

\* Please Note: To reduce risk of needle stick injury the pigtail straightening device can be removed by sliding it to the base of the catheter & peeling away. Dipping or wiping the Neo-Hydro in saline solution will activate the hydrophilic coating & facilitate smooth insertion.

\*\* Please Note: If catheter placement is not correct at this stage the stylet can be re-introduced into the cannula and the system repositioned.